



The suggested protocol for [Psoriasis](#) with the CellSonic VIPP is:

Energy level 2 with the infinity head,

Apply gel in the usual way and then 50 shots/cm² on treated skin area,

4 treatments in total. Minimum time between treatments 2 weeks.

No drugs. No side effects. The patient's preferred creams can continue to be used.

Pictures below show the condition on the 8th November 2015 with the picture to the right showing the same area on 6th December 2015 after only one treatment.

The terrible itching and pain of the psoriasis was removed straight away and the patient remained as good as itch-free afterwards.

Reports from patients treated more than a year ago are encouraging because the area treated will eventually clear and remains clear.

However, in another area the psoriasis can break out and then that area has to be treated.

It could be that the treatment of psoriasis is affecting a systemic condition. There was a case of psoriasis on both shoulders so only one side was treated and then both sides cleared. Areas left untreated cleared quickly when later they were treated. This suggests the CellSonic VIPP is affecting the response of the immune system; something we observe in healing chronic wounds and repairing nerves.





www.cellsonic-medical.com